## ENGLISH

### About myAir

The myAir mobile application is a patient self-monitoring software platform which allows you to:

- Connect and disconnect your smartphone with your AirSense™ 11 device using **Bluetooth®** wireless technology.
- Access a step-by-step therapy acclimatisation tutorial; this is known as **Test Drive**. This feature controls your device and allows you to trial pressures to simulate therapy.
- Troubleshoot mask fitting issues.
- Access your personal patient data such as your usage and reports on therapy progress.
- View tailored educational and coaching advice such as therapy tips and information Obstructive Sleep Apnea (OSA).

### CAUTION

In the US, Federal law restricts this device to sale by or on the order of a physician.

### Indications for use

The myAir app is intended for patients who are prescribed a compatible ResMed AirSense 11 device to simulate therapy prior to using their device with their prescribed settings. It is an optional software accessory to allow patients to acclimate to their therapy device.

### Intended use

myAir is a patient engagement software platform for use by patients who are prescribed a compatible ResMed device for the purpose of self-tracking therapy usage data and receiving coaching (educational videos, tips) in a personal home setting. The Personal Therapy Assistant (PTA) feature of myAir is intended for patients who are prescribed a compatible ResMed Air11 platform device to remotely simulate therapy prior to using their device with their prescribed settings. myAir is an optional software accessory to allow patients to acclimate to their therapy device.

### Clinical benefits

The myAir app is a Patient Engagement application which supports patients in managing their Sleep-Disordered Breathing (SDB). The App provides patients with educational tools, promotes positive behavior change, and allows patients to track their progress.

### System requirements

- iOS™ 14.0 and later
- Android™ 8.0 and later.

### Compatible ResMed devices

- AirSense 11

## Other ResMed devices<sup>a</sup>

- AirSense 10
- AirCurve 10

**Note:** Bluetooth features are only available on AirSense 11. Not all compatible devices are available in all regions.

<sup>a</sup> Other ResMed devices do not directly connect to or directly exchange data with the myAir app. The myAir app accesses therapy data indirectly through ResMed's secure cloud database, AirView.

## Security

Built-in security features, steps you take to protect your smart device against malware, and keeping up with software and firmware updates all help to safeguard your privacy, protect your smart device and ensure optimal performance.

The myAir app uses security features to help protect your privacy and prevent unauthorised access to your data. Security features include:

- Unique keys for Bluetooth pairing
- Application-level encryption for the transmission of data, in addition to Bluetooth security
- Data encryption

Protecting your smart device against malware helps keep your personal information and sleep data safe and secure. ResMed recommends the following:

- Use a strong account password and do not share it with anyone else.
- Enable personal identification number (PIN) or fingerprint security on your smart device. Refer to your smart device user instructions for information on enabling either feature.
- Avoid unsafe modification of the smart device's operating system.
- Consider enabling remote data wipe on your smart device. Remote data wipe is a function available on your smart device for remotely erasing personal data in the event that your device is lost or stolen. Refer to your smart device user instructions for information on enabling this feature.
- Keep your operating system up to date with security patches.

## Setting up Bluetooth Connection with the AirSense 11 device

Make sure your AirSense 11 device is powered on. For instructions on how to setup and power on your device, refer to the AirSense 11 user guide.

### If you are a first time myAir user:

1. In the myAir app, follow the prompts on the app to create an account.
2. To select your device, tap **AirSense 11**.
3. Follow the prompts on the app to complete the Bluetooth connection.

### If you have an existing myAir account:

1. In the myAir app, tap **Profile**.
2. Tap **My equipment**.
3. Tap **Pair Bluetooth**.
4. Follow the prompts on the app to complete the Bluetooth connection.

The AirSense 11 device is now connected to the myAir app. The Bluetooth connection symbol appears on the AirSense 11 device status bar to confirm the connection between the device and the smartphone. Tap **Done** to finish.

## Disconnecting Bluetooth

1. In the myAir app, tap **Profile**.
2. Tap **My equipment**.
3. Tap **Update device**.
4. Tap **Disconnect**.
5. To exit the page, tap the back arrow icon.

## Test Drive

Test Drive gives you an opportunity to try out what therapy feels like. It is designed to slowly acclimatise you to therapy by introducing you to low (4 cmH<sub>2</sub>O) pressure before moving you to higher (7 and 10 cmH<sub>2</sub>O) pressure. This feature requires an active Bluetooth connection to your AirSense 11 device and your mask information.

Make sure you have an active Bluetooth connection. If Bluetooth connection is inactive, see **Setting up Bluetooth Connection with the AirSense 11 device**.

1. In the myAir app, tap **Coaching**.
2. Tap **Library**.
3. Tap **Test Drive**.
4. Tap the **Let's go** button.
5. Follow the on-screen prompts, Test Drive will slowly introduce you to higher pressure.

## myAir score

myAir calculates your score by analyzing your nightly therapy data. The higher your score, the better. You get points based on the following four key categories.

### Usage hours

The point system for usage is calculated in hours and minutes. If you use your therapy for 1 hour you get 10 points, or for 2.3 hours (2 hours, 18 minutes) you get 23 points. The more time you use your therapy, the more points you receive, up to a maximum of 70 points.

### Mask seal

The better your mask seal, the more points you get. This category can help you know if you need to adjust or change your mask to get a better fit. If your mask seal is poor, it can affect your comfort and the quality of your treatment. Your score reduces as your mask leak increases. You can get up to 20 points for minimal mask leak, 10 to 15 points for moderate leak, and 0 to 10 points for higher leak.

### Events per hour

The fewer breathing events you have each hour, the more points you get. These breathing events are also known as the apnea-hypopnea index (or AHI). myAir measures how many times your breathing partially or fully stops each hour. If you have minimal events, you get 4 to 5 points.

### Mask on/off

The fewer times you take your mask on and off throughout the night, the more points you get. Everyone has to take their mask on and off one time during treatment. So, for example, if you remove your mask one or two times, you get 5 points. However, if you take your mask on and off several times, it can indicate a problem with mask fit or with your sleep in general.

## Mask fitting issues

Throughout the Test Drive process, myAir will monitor for mask air leak issues. In the event that a mask air leak issue is detected, myAir will display mask troubleshooting tips specific to your mask.

Follow the on-screen prompts and instructions.

Once myAir detects the mask air leak issue has been resolved, you will be notified and returned to Test Drive.

Additionally, myAir will notify the user if Test Drive failed due to a device error. To understand and resolve the error, please refer to the front panel of your device and the AirSense 11 user guide.

## Troubleshooting

If you experience any problems, try the following suggestions. Also refer to your AirSense 11 user guide. If you are not able to fix the problem, contact your care provider or ResMed.

If you experience problems with the AirSense 11 device, do not open it.

Problem/possible cause	Solution
<b>I cannot establish a Bluetooth connection.</b>	
Bluetooth may be disabled.	Check your smartphone to determine whether Bluetooth is enabled   Reconnect and pair your AirSense 11 device and smartphone. Refer to <b>Setting up Bluetooth Connection with the AirSense 11 device</b> for more information.  If these solutions do not resolve the issue, try restarting your smartphone.
Your Bluetooth connection may be poor.	Ensure that your AirSense 11 device is placed away from potential sources of interference such as microwaves, wireless local area network (WLAN) routers or cordless phones.
Your device may be in airplane mode.	Turn off <b>Airplane Mode</b> .  For instructions, refer to your AirSense 11 user guide.
The AirSense 11 device appears in the device list, but I cannot establish a connection.	Tap the device name in the device list to confirm that you want to connect to that device.

## Further information

For information on cleaning your AirSense 11 device, maintenance, troubleshooting and other technical specifications, refer to your AirSense 11 user guide.

**ResMed Corp**  
9001 Spectrum Center Blvd  
San Diego, CA 92123  
USA

**DISTRIBUTED BY**  
ResMed Pty Ltd  
1 Elizabeth Macarthur Drive  
Bella Vista, NSW 2153  
Australia

**ResMed SAS**  
Parc Technologique de Lyon  
292 Allee Jacques Monod  
69791 Sain Priest Cedex  
France

See ResMed.com for other ResMed locations worldwide. AirSense, Air11, AirCurve and myAir are trademarks and/or registered trademarks of the ResMed family of companies. The Bluetooth® wordmark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by ResMed is under license. iOS is a trademark or registered trademark of Cisco in the U.S. and other countries and is used under license. Android is a trademark of Google LLC. For Patent and other intellectual property information, see ResMed.com/ip. © 2022 ResMed. 398127/5 2022-05

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